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Product data sheet AllegraMask®

1. Intended use / areas of application

FAIST AllegraMask® is used to protect against infectious diseases caused by the transmission of droplets and liquids (droplet infections). The AllegraMask® creates a tight barrier between the patient or user during normal breathing and leads to a reduction in the transfer of droplets (external effect through spitting protection). This can reduce (not prevent) the risk of disease transmission. If necessary, an MNS mask (surgical mask) can also be worn under the AllegraMask® in order to further reduce the risk of infection.

FAIST AllegraMask® can also reduce, prevent and alleviate the transmission and symptoms of illnesses that occur through touching the nose and mouth, for example by protecting the transmission of bacteria, viruses and fungi through touch (self-protection against smear infection).

AllegraMask® can also be used in all applications in which the user / patient cannot wear an operating mask or other MSM for health reasons (for example, because the respiratory tract is too long and reduced).

AllegraMask® can also be used in all applications in which the facial expressions of the user / patient play an important role, for example at work or from psychological social applications. For example, also for the deaf or for people with physical disabilities.

AllegraMask® can also be used in all applications where face protection is required to prevent and alleviate skin diseases on the face, such as acne.

Due to its design, full protection against viruses, bacteria or other biological agents can only be given to a limited extent and therefore cannot be guaranteed.

AllegraMask® is used in the private, service, medical, nursing, rescue and other fields.

2. Functional description

The mask is put on like a pair of glasses over the nose and mouth area. The fit should completely cover the mouth and nose area. If this is not the case, the mask should not be used and the manufacturer should be contacted. The AllegraMask® should be cleaned when used for the first time.

3. Safety instructions / contraindications

For your own safety and that of third parties, please note the following:

- ATTENTION: Please clean the before using it for the first time
- Only original spare parts may be used, otherwise there may be restrictions on function or biocompatibility.
- ATTENTION: Not suitable for small children, including because of small parts that can be swallowed
- ATTENTION: Not suitable for use in the surgical field and similar areas of application (no surgical mask). In these cases, the AllegraMask® can be used as additional protection.
- No complete protection against infectious diseases through the transmission of droplets and aerosols (droplet infection). To avoid, prevent and reduce infectious diseases, please also take additional measures such as keeping your distance and adequate hand hygiene.
- To avoid infection or bacterial contamination, please refer to the "Cleaning and disinfection" section
- ATTENTION: The mask is delivered with a tether. Please note the applicable regulations for your use, UVV / work on machines, etc. If there is a strong train, there may be an accident. Please only attach the tether so that it detaches from the bracket when you pull gently.
- Harmlessness: The materials used are not known to be harmful to the user, cause skin irritation or breathing difficulties. However, materials that come into contact with the skin of the wearer may cause allergic reactions in susceptible people. In these cases, the mask should not be used and the manufacturer should be contacted.
- If you feel uncomfortable after wearing the mask for a long time, remove the mask, stop using it and contact the manufacturer.
- The AllegraMask® offers no protection against mechanical influences, impacts, splinters, mechanical parts and the like.
- The AllegraMask® should not be used or replaced if it is scratched, cracked or otherwise damaged, shows changes in shape and color, has been exposed to chemicals, oils or greases, has been exposed to extreme heat or has been exposed to the sun for a long time. Damage may not be visible to the end user.
- When using anti-fog and anti-static agents: When using, only the agent recommended by the manufacturer should be used. In this case, also observe the safety instructions from the anti-fog manufacturer.
- It is recommended to wear the AllegraMask® for a maximum of two hours without a break.

4. Cleaning and disinfection

Please clean or disinfect the mask after each use. When changing patients, the mask must be disinfected.

4.1.Cleaning

Wash all parts in warm water using a soap solution or alkaline detergent; then rinse under warm running water to remove any residue. Hard brushes, abrasives or other materials that can damage the surfaces must not be used.

4.2. Disinfection

All parts made of PVC, silicone or polycarbonate can be disinfected using commercially available disinfectants that are suitable for plastic materials. The manufacturer's information on exposure times and application concentrations of the disinfectants must be followed. Please also observe the safety instructions for the disinfectant from the manufacturer.

5. Anti-fog and anti-static agents

Anti-fog and anti-static agents can be used as a useful addition. When using, only the agent specified by us is to be used. In this case, also observe the manufacturer's safety instructions.

6.Technical data / storage / transport

Storage temperature: - 40 ° C to 60 ° C

Operating temperature: -18 ° C to 50 ° C

Duration of use: It is recommended to wear the AllegraMask® for a maximum of two hours without a break.

Storage: Storage in a laundry or cotton bag is recommended. Do not leave it in the sun.

7. Materials

Plastic components can contain phthalates (see labeling). Take special care with: newborns • children before puberty • breastfeeding women • pregnant women

1	Bracket	MABS
2	Nose pad	SEBS
3	Visor	PC
4	Strap	Cotton

8. Warranty

1-year warranty in accordance with our guarantee provisions: - We grant material or manufacturing defects from the date of purchase only in connection with the proof of purchase.

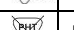
9. Spare parts / accessories

- Bracket
- protective shield
- strap
- nose pad
- Anti-fog VisioClear, Bosch (please note the manufacturer's instructions for use)

10. Disclaimer of liability

To the best of our knowledge, the information above is correct. However, it is not claimed that these are complete and it should therefore only be considered as a guideline. The information provided here is intended to describe the product with regard to the necessary safety precautions, it is not intended to guarantee certain properties, is based on the current state of our knowledge and does not constitute a contractual legal relationship. The company cannot be held responsible for any damage caused by handling or contact with the above product. Liability excluded.

11. Explanation of symbols

	Latex free		Protect from heat and light
	Not for reuse		Order number
	Follow the instructions		Batch name
	Best before [date		Contains phthalate
	Manufactured on		Phthalate free

FAIST Systeme GmbH
Daimlerstr. 3 | D - 86836 Untermeitingen
E-Mail: allegramask@FAISTlightmetals.com | www.allegramask.com
Fon +49 (0) 8232 9638 47 | Fax +49 (0) 8232 9638 24
Conformity according to EC directive 93/42 / EEC Annex VII

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